



GORE VIATORR®

TIPS Endoprosthesis


KEY IMPLANTATION STEPS

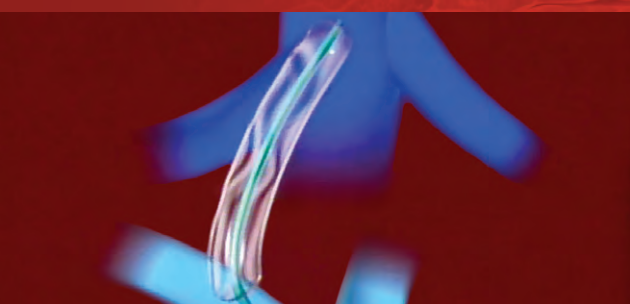

Pre-dilate tract. An undersized balloon may be used to enhance tactile feel.

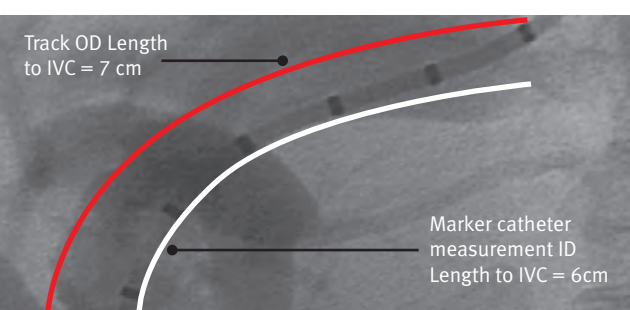

Measure tract length using marker catheter. Add 1 cm to measurement for proper graft-lined length selection.

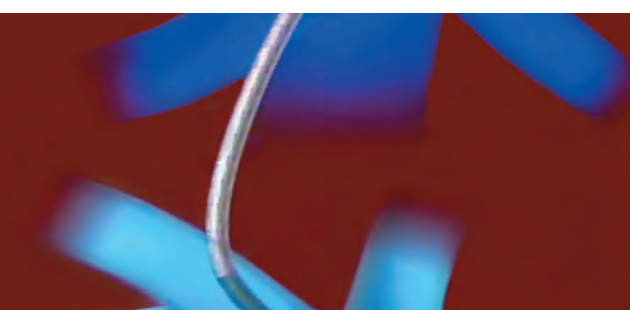


Advance sheath \geq 3 cm into portal vein.



Flush system with finger covering tip of clear access sleeve. Do not remove clear access sleeve.



Insert clear access sleeve into hemostasis valve until there is a tactile stop. Maintain forward pressure on access sleeve during catheter advancement.

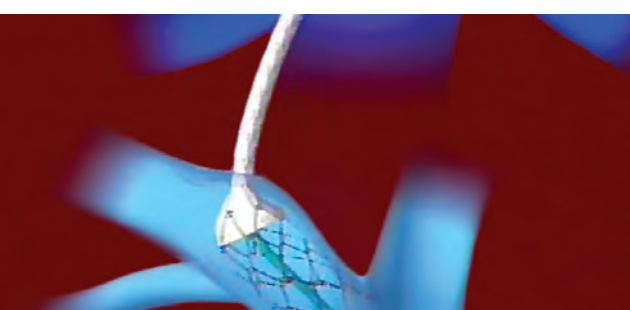


De Novo Procedures

Pull sheath back to IVC to deploy bare stent portion. Gently pull endoprosthesis back until resistance is felt at the portal / parenchymal junction.

Revision Procedures

Align gold band with portal / parenchymal junction prior to deployment of bare stent portion.

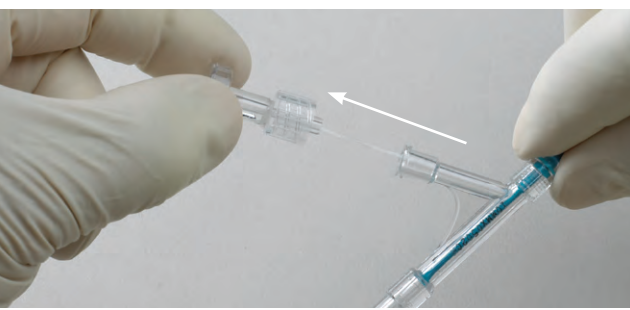

Deploy covered stent portion using GORE SIM-PULL Delivery System.

Note: Follow-up ultrasound should be performed 72 hours post-procedure for best results.

SIZING TABLE

| ENDOPROSTHESIS DIMENSIONS | | | | | | RECOMMENDED ACCESSORY EQUIPMENT | | |
|---------------------------|---|-------|-------|-------|-------|---------------------------------|-----------------------------------|--|
| INTERNAL DIAMETER (mm) | GRAFT-LINED LENGTH / UNLINED LENGTH (cm / cm) | | | | | MAXIMUM GUIDEWIRE DIAMETER (in) | HEMOSTATIC INTRODUCER SHEATH (Fr) | MAXIMUM DILATATION BALLOON DIAMETER (mm) |
| | Labeled | 4 / 2 | 5 / 2 | 6 / 2 | 7 / 2 | | | |
| 8 | x | x | x | x | x | \leq 0.038 | 10 | 8 |
| 10 | x | x | x | x | x | \leq 0.038 | 10 | 10 |
| 12 | x | | x | | x | \leq 0.038 | 10 | 12 |

Consult Instructions for Use

INDICATIONS FOR USE: The GORE VIATORR® TIPS Endoprosthesis is indicated for use in the *de novo* and revision treatment of portal hypertension and its complications such as variceal bleeding, gastropathy, refractory ascites, and / or hepatic hydrothorax. Refer to the *Instructions for Use* at goremedical.com for contraindications, warnings and precautions. Only

W. L. GORE & ASSOCIATES, INC.

Flagstaff, AZ 86004

+65.67332882 (Asia Pacific)

00800.6334.4673 (Europe)

800.437.8181 (United States)

928.779.2771 (United States)

goremedical.com

Products listed may not be available in all markets.
GORE, SIM-PULL, VIATORR®, and designs are trademarks of W. L. Gore & Associates.
© 2006, 2007, 2009 W. L. Gore & Associates, Inc. AN0669-EN1 SEPTEMBER 2009



PERFORMANCE
by design