INSTRUCTIONS FOR USE FOR:

PANTALOON

PROTECTIVE LINER

en

English

nl

Nederlands

fr

Français

de

Deutsch

it

Italiano

es

Español

MEDICAL

INSTRUCTIONS FOR USE FOR PANTALOON Protective Liner

DESCRIPTION

PANTALOON Protective Liner is a waterproof, breathable membrane manufactured from expanded polytetrafluoroethylene. PANTALOON Protective Liner protects the cast and padding from becoming soiled by urine and excrement. Urine and excrement may be removed from under the PANTALOON Protective Liner, thus simplifying cleaning and improving patient hygiene.

INDICATIONS

PANTALOON Protective Liner should be used in body cast applications where the physician determines the need for a barrier to urine and excrement.

PRECAUTIONS

Contaminants should not be introduced into the cast, including: oils, oil based lotions, powders, sand, dirt, grease or solvents.

Foreign objects should not be inserted into the cast. Foreign objects can tear the PANTALOON Protective Liner, compromising the effectiveness of the barrier to urine and excrement.

Some patients may experience standard cast problems including: soiled padding, odor, skin irritation, maceration, and rash.

Patients should not be immersed in water with casts made with plaster casting tape or cotton and synthetic padding material.

INSTRUCTIONS FOR HOME CARE

Patient/care-giver should be instructed in general cast care and when to contact their doctor.

Contaminants that could cause skin irritation, such as oils, oil based lotions, powders, sand, dirt, grease, or solvents should not be allowed to enter the cast.

If such a contaminant does enter the cast, the attending health care professional should be contacted. Foreign objects should not be placed inside the cast.

STERILIZATION (NOT STERILE)

PANTALOON Protective Liner is provided non-sterile. Sterilization of this product is not recommended.

APPLICATION GUIDE

1. To unfold the PANTALOON Protective Liner, lift it by holding the edge of the garment with the marker on it. The Liner unfolds by itself as you lift. Gather the material with your fingers, beginning at the torso and continuing down one leg. Slip patient's foot through this opening. Repeat process for other leg.

2. Extend PANTALOON Protective Liner down the legs and over chest, leaving sufficient material for creating cuffs. Align perineal opening on the Liner to the patient's perineal area. Gather Liner in the perineal area to ensure sufficient material to create perineal cuff.

Note: Liner is loose fitting. Folds and wrinkles are of no clinical significance.

3. Place patient on the spica table at this point.

Before applying cast padding, place an abdominal towel under the Liner as you would previous to applying any hip spica cast.

Apply cast padding over Liner. Allow sufficient Liner for cuffing at torso and legs.

4. Apply fiberglass casting tape (or plaster) over cast padding.

5. After cutting out perineal opening with a cast saw, leave sufficient Liner and cast padding to create cuffs.

6. Apply casting material to secure cuffs. Create similar cuffs at all cast ends.

Note: The Liner can best protect the underlying cast padding when you create a cuff that completely seals in the perineal area.

7. After the cast has thoroughly dried, if desired, petal the perineal opening with waterproof surgical tape. Petaling reinforces the seal and further protects the padding.

GUARANTEE

Your satisfaction with performance and value is guaranteed.

- If for any reason you are unable to use a GORE Casting Product, we will replace it free of charge.
- If for any reason you are not satisfied with any GORE Casting Product, contact us for a full refund at 800-528-8763 or 520-526-3030.

DEFINITIONS

 Attention, See Instructions for Use

 Do Not Reuse

 Batch Code

 Catalogue Number

 European Authorized Representative

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© 1999 W. L. Gore & Associates, Inc.
GORE, PANTALOON, and designs are trademarks of W. L. Gore & Associates.
W. L. GORE & Associati S.r.l.
W. L. GORE & Associates, Inc.