



GORE® VIATORR® TIPS Endoprosthesis


KEY IMPLANTATION STEPS

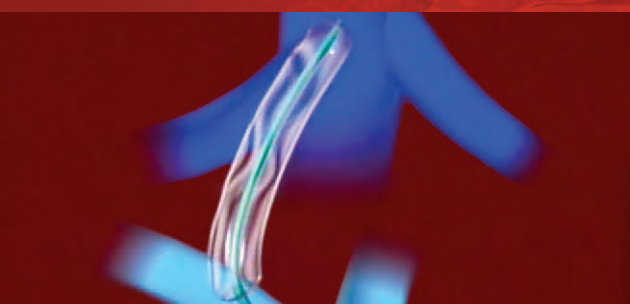

Pre-dilate tract. An undersized balloon may be used to enhance tactile feel.

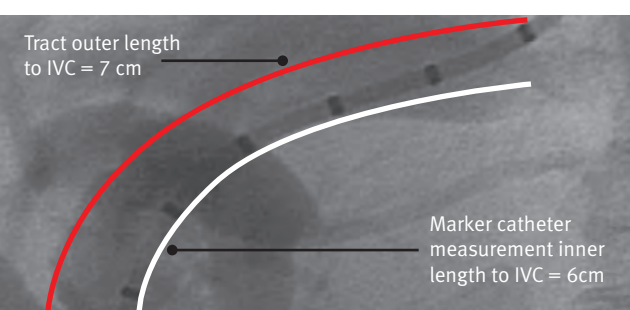

Measure tract length using marker catheter. Add 1 cm to measurement for proper graft-lined length selection.

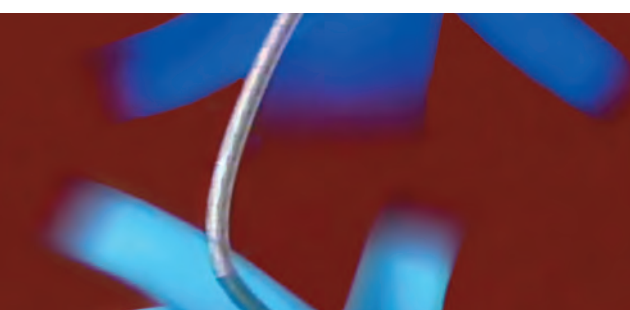

De Novo Procedures

Advance sheath \geq 3 cm into portal vein.

Revision Procedures


Advance sheath to distal end of stent being revised.



Flush system with finger covering tip of clear access sleeve. Do not remove clear access sleeve.



Insert clear access sleeve into hemostasis valve until there is a tactile stop. Maintain forward pressure on access sleeve during catheter advancement.

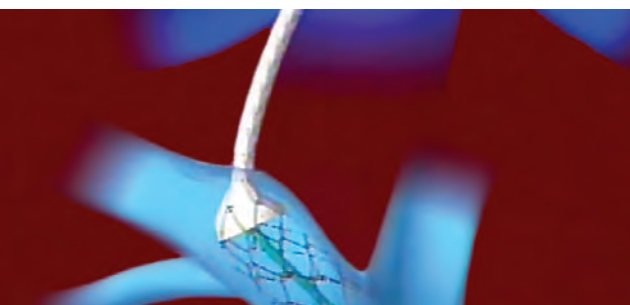

De Novo Procedures

Pull sheath back to IVC to deploy bare stent portion. Gently pull endoprosthesis back until gold marker is just distal to the portal / parenchymal junction.

Revision Procedures

Align gold marker with portal / parenchymal junction prior to deployment of bare stent portion.



Deploy covered stent portion using GORE SIM-PULL Delivery System.

Note: It is recommended that Doppler ultrasound be performed 72 hours post-TIPS.

SIZING TABLE

ENDOPROSTHESIS DIMENSIONS						RECOMMENDED ACCESSORY EQUIPMENT		
INTERNAL DIAMETER (mm)	GRAFT-LINED LENGTH / UNLINED LENGTH (cm / cm)					MAXIMUM GUIDEWIRE DIAMETER (in)	HEMOSTATIC INTRODUCER SHEATH (Fr)	MAXIMUM DILATATION BALLOON DIAMETER (mm)
	Labeled	4 / 2	5 / 2	6 / 2	7 / 2			
8	x	x	x	x	x	\leq 0.038	10	8
10	x	x	x	x	x	\leq 0.038	10	10
12*	x		x		x	\leq 0.038	10	12

* Not available in all markets

Consult Instructions for Use

INDICATIONS FOR USE IN THE US: The GORE® VIATORR® TIPS Endoprosthesis is indicated for use in the *de novo* and revision treatment of portal hypertension and its complications such as variceal bleeding, gastropathy, refractory ascites, and / or hepatic hydrothorax. **INDICATIONS FOR USE UNDER CE MARK:** The GORE® VIATORR® TIPS Endoprosthesis is indicated for use in the treatment of portal hypertension and its complications such as: variceal bleeding refractory to, or intolerant of, conventional therapies, inaccessible varices, gastropathy, refractory ascites, and / or hepatic hydrothorax. Refer to *Instructions for Use* at goremedical.com for a complete description of all contraindications, warnings, precautions and adverse events. Rx Only

W. L. GORE & ASSOCIATES, INC.
Flagstaff, AZ 86004

+65.67332882 (Asia Pacific)
00800.6334.4673 (Europe)
800.437.8181 (United States)
928.779.2771 (United States)

goremedical.com

Products listed may not be available in all markets.
GORE®, PERFORMANCE BY DESIGN, SIM-PULL, VIATORR®, and designs are trademarks of W. L. Gore & Associates.
© 2006, 2007, 2009, 2010 W. L. Gore & Associates, Inc. AN0669-EN2 NOVEMBER 2010



PERFORMANCE
by design