


GORE® VIABAHN® VBX
Balloon Expandable Endoprosthesis

SIZING TABLE

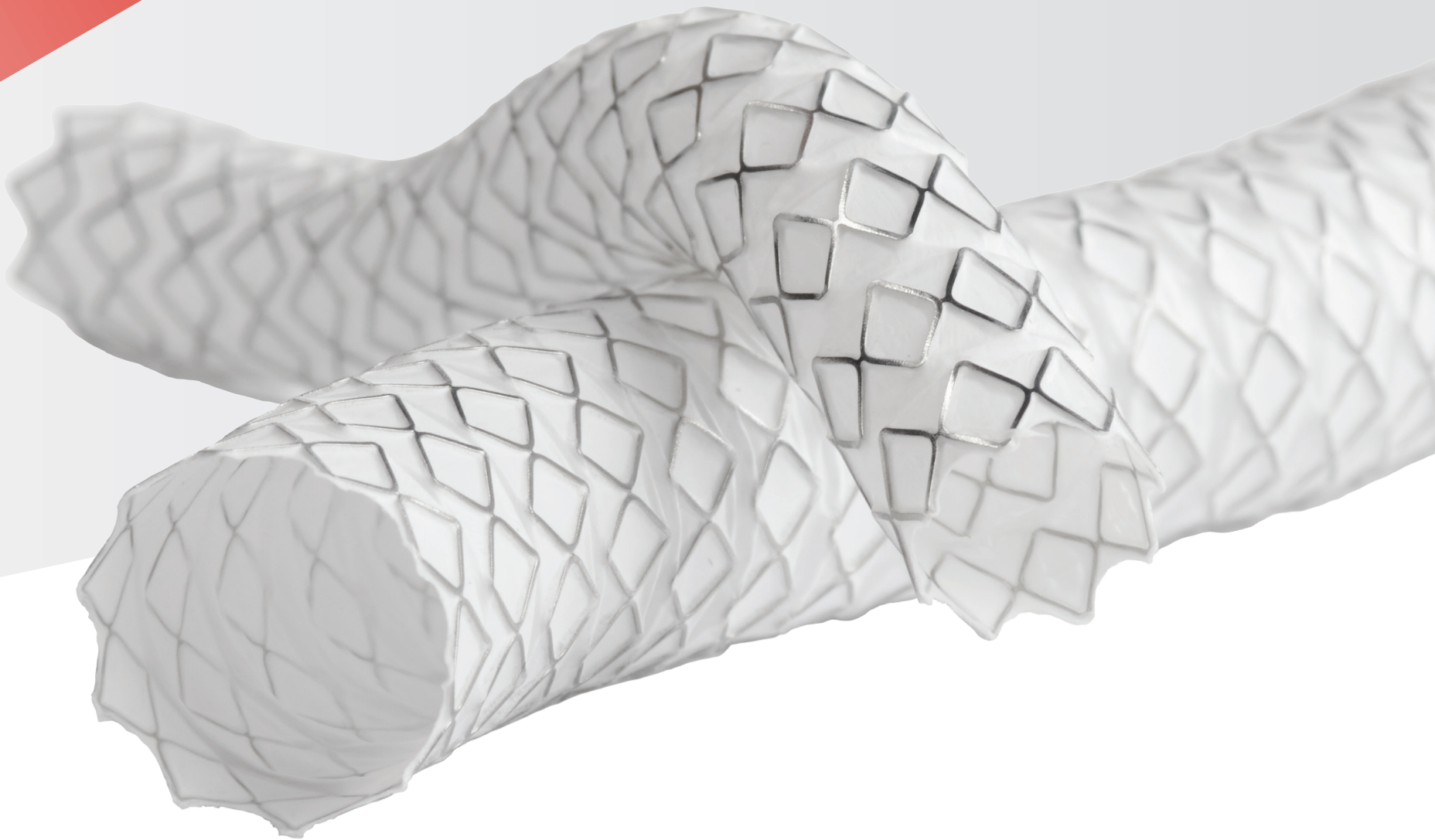

Together, improving life



GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis

Sizing table

Stent labeled/ nominal diameter (mm)	Crimped stent length (mm)	Introducer sheath size (Fr)	At minimal pressure			At nominal pressure			At rated burst pressure			At max. post-dilated stent diameter*	
			Minimal inflation pressure (atm/kPa)	Final stent ID (mm)	Stent length† (mm)	Nominal inflation pressure (atm/kPa)	Final stent ID (mm)	Stent length† (mm)	Rated burst pressure (atm/kPa)	Final stent ID (mm)	Stent length† (mm)	Max. post-dilated diameter (mm)	Stent length at max. post-dilated stent diameter† (mm)
5	15	7	7/709	4.1	14.1	14/1418	5.0	13.6	15/1520	5.1	13.5	8	10.5
	19	7	7/709	4.3	18.5	12/1216	5.0	18.2	15/1520	5.3	18.0	8	13.8
	29	7	7/709	4.4	28.0	12/1216	5.0	27.6	15/1520	5.3	27.3	8	22.3
	39	7	7/709	4.5	38.1	11/1114	5.0	37.8	15/1520	5.4	37.5	8	32.0
	59	7	7/709	4.5	58.2	11/1114	5.0	57.7	15/1520	5.3	57.3	8	51.2
	79	7	6/608	4.3	77.9	12/1216	5.0	77.3	15/1520	5.3	76.9	8	72.1
6	15	7	7/709	4.8	14.0	15/1520	6.0	13.1	15/1520	6.0	13.1	8	10.5
	19	7	7/709	4.9	18.2	15/1520	6.0	17.4	15/1520	6.0	17.4	8	14.6
	29	7	7/709	5.2	28.1	13/1317	6.0	27.5	15/1520	6.2	27.3	8	24.3
	39	7	7/709	5.0	38.1	14/1418	6.0	37.4	15/1520	6.1	37.3	8	33.3
	59	7	7/709	5.1	58.2	14/1418	6.0	57.5	15/1520	6.1	57.4	8	52.3
	79	7	7/709	5.3	77.6	12/1216	6.0	77.0	15/1520	6.3	76.7	8	73.6
7	15	7	6/608	5.6	13.9	15/1520	6.9	13.4	15/1520	6.9	13.4	11	9.5
	19	7	7/709	6.1	18.1	11/1114	6.9	17.7	15/1520	7.4	17.3	11	12.6
	29	7	5/507	5.9	28.0	11/1114	7.0	27.3	15/1520	7.4	26.8	11	19.4
	39	7	6/608	6.0	37.9	12/1216	7.0	37.0	15/1520	7.3	36.5	11	26.6
	59	7	6/608	6.0	57.8	12/1216	7.0	57.1	15/1520	7.3	56.7	11	46.9
	79	7	6/608	6.0	78.0	14/1418	7.0	77.3	15/1520	7.1	77.2	11	68.1
8	29	7	6/608	7.0	27.5	12/1216	8.0	26.9	13/1317	8.1	26.8	11	23.7
	39	7	6/608	6.7	37.1	12/1216	8.0	36.2	13/1317	8.1	36.1	11	30.0
	59	7	6/608	6.8	56.9	12/1216	8.0	56.1	13/1317	8.1	56.0	11	47.3
	79	8	6/608	7.0	77.2	11/1114	8.0	76.6	13/1317	8.3	76.4	11	69.6
8L	29	7	5/507	6.7	28.7	12/1216	8.0	29.0	13/1317	8.1	29.0	16‡	21.8
	39	7	5/507	6.7	38.6	11/1114	8.0	39.0	13/1317	8.3	39.2	16‡	26.8
	59	8	5/507	6.7	58.4	11/1114	8.0	59.4	13/1317	8.3	59.7	16‡	44.1
	79	8	5/507	6.7	78.5	12/1216	8.0	79.3	13/1317	8.1	79.4	16‡	58.6
9	29	8	5/507	7.7	27.3	11/1114	9.0	26.7	13/1317	9.3	26.4	13	20.4
	39	8	7/709	8.0	36.8	12/1216	9.0	36.3	13/1317	9.1	36.2	13	23.2
	59	8	6/608	7.8	56.9	11/1114	8.9	56.0	13/1317	9.2	55.7	13	38.1
	79	8	5/507	7.8	76.9	10/1013	8.9	76.2	13/1317	9.4	75.8	13	62.7
10	29	8	5/507	8.7	25.7	10/1013	10.0	25.4	13/1317	10.4	25.2	13	19.1
	39	8	6/608	8.8	35.7	11/1114	9.9	35.4	13/1317	10.2	35.3	13	31.1
	59	8	5/507	8.6	55.5	11/1114	10.0	55.3	13/1317	10.3	55.2	13	44.6
	79	8	5/507	8.6	75.7	11/1114	10.0	76.1	13/1317	10.3	76.2	13	68.4
11	29	8	5/507	9.0	26.6	10/1013	10.9	25.7	12/1216	11.4	25.4	16‡	17.8
	39	8	5/507	9.4	35.5	11/1114	11.0	34.7	12/1216	11.2	34.6	16‡	25.8
	59	8	5/507	9.5	55.7	10/1013	11.0	54.9	12/1216	11.4	54.5	16‡	42.8
	79	8	5/507	9.5	74.3	10/1013	11.0	73.9	12/1216	11.5	73.7	16‡	60.6

* Secondary balloon required to post-dilate the stent beyond its nominal deployed diameter (secondary balloon not included).
† Based on idealized in-vitro conditions and provided to be used as a general guide.
‡ Technical limit of the device as determined by in-vitro testing for the indicated use — see *Instructions for Use*.

Visit goremedical.com/ap



Refer to *Instructions for Use* for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. **Rx Only**

Products listed may not be available in all markets.

GORE, *Together, improving life*, VBX, VIABAHN and designs are trademarks of W. L. Gore & Associates.
© 2022 W. L. Gore & Associates, Inc. 21407192-EN JANUARY 2022

W. L. Gore & Associates, Inc.
goremedical.com

Asia Pacific +65 6733 2882 Australia/New Zealand 1800 680 424 Europe 00800 6334 4673
United States Flagstaff, AZ 86004 800 437 8181 928 779 2771

