



Together, improving life

GORE® Tri-Lobe Balloon Catheter

INFLATION VOLUME GUIDE

BCM1634 – 10 mm Balloon Catheter

Intended aortic diameter (mm)	Recommended inflation volume (ml)
16	4
20	6
24	8
28	10
32	13

BCL2645 – 16 mm Balloon Catheter

Intended aortic diameter (mm)	Recommended inflation volume (ml)
26	12
30	14
34	18
38	21
42	25



 Consult Instruction
for Use
eifu.goremedical.com

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. Rx Only

Products listed may not be available in all markets.

GORE, *Together, improving life*
and designs are trademarks of
W. L. Gore & Associates.
© 2021 W. L. Gore & Associates, Inc.
2148931-EN JANUARY 2021

