


## MAKE EARLY CANNULATION COUNT

Three steps for success for the first three sessions within two weeks of graft implantation:

### STEPS TO PREP

- Clean gown
- Mask or shield
- Sterile gloves

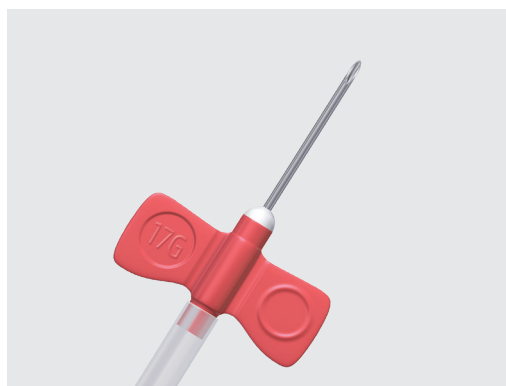

1

### GO SMALL


Use a 17 gauge needle to restrict flow



2

### WATCH FLOW

Keep the flow rate at 200–250 ml/min to prevent damage to the vein

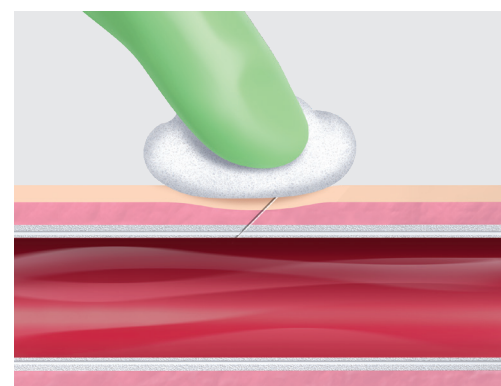

3

### HOLD PRESSURE

Hold constant pressure for 10–15 minutes to achieve hemostasis

 Consult Instructions for Use  
[eifu.goremedical.com](http://eifu.goremedical.com)

Products listed may not be available in all markets.

GORE, *Together, improving life*, ACUSEAL and designs are trademarks of W. L. Gore & Associates.  
© 2021 W. L. Gore & Associates, Inc. 21259592-EN AUGUST 2021

Refer to *Instructions for Use* at [eifu.goremedical.com](http://eifu.goremedical.com) for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. <sup>®</sup> Only

W. L. Gore & Associates, Inc.  
[goremedical.com](http://goremedical.com)


Asia Pacific +65 67332882 Australia/New Zealand 1800 680 424 Europe 00800 6334 4673  
United States Flagstaff, AZ 86003 800 437 8181 928 779 2771

*Together, improving life*



## EARLY CANNULATION AND UNCOMPROMISED HANDLING

Three key steps for implantation:

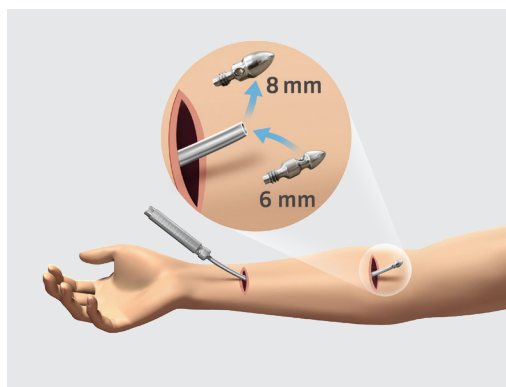

1

### CREATE

**6 mm graft:** tunnel with 8 mm tip, attach graft with 6 mm tip


**4–7 mm graft:** tunnel with 10 mm tip, attach graft with 7 mm tip



2

### CUT

Different tools create different cuts. Scissors are preferred and recommended.

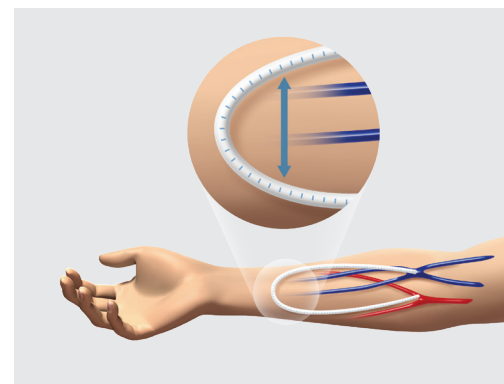

3

### SUTURE

Position the graft with a wide curve at the apex

Consult Instructions  
for Use  
[eifu.goremedical.com](http://eifu.goremedical.com)

Products listed may not be available in all markets.

GORE, *Together, improving life*, ACUSEAL and designs are trademarks of W. L. Gore & Associates.  
© 2021 W. L. Gore & Associates, Inc. 21259592-EN AUGUST 2021

Refer to *Instructions for Use* at [eifu.goremedical.com](http://eifu.goremedical.com) for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available.  Only

W. L. Gore & Associates, Inc.  
[goremedical.com](http://goremedical.com)

Asia Pacific +65 67332882 Australia/New Zealand 1800 680 424 Europe 00800 6334 4673  
United States Flagstaff, AZ 86003 800 437 8181 928 779 2771

*Together, improving life*

