

GORE® EXCLUDER®

Thoracoabdominal Branch
Endoprosthesis (TAMBE)

Device sizing pocket reference guide

See *Instructions for Use (IFU)*
for complete information.




Together, improving life




Aortic Component

GORE® EXCLUDER® Thoracoabdominal Branch Endoprosthesis

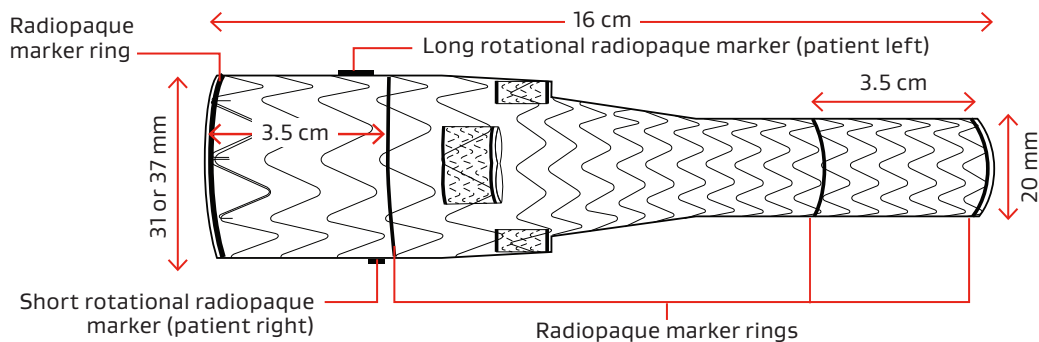


Catalogue number	Proximal device diameter (mm)	Intended proximal aortic diameters (mm)	SMA/CA internal portal diameter (mm)	Renal internal portal diameter (mm)	Distal device diameter (mm)	Overall device length (cm)	Catheter working length (cm)	Recommended GORE® DRYSEAL Flex Introducer Sheath (Fr)
ATAA43120160A	31	22-29	8	6	20	16	66	22
ATAA43720160A	37	27-34	8	6	20	16	66	22

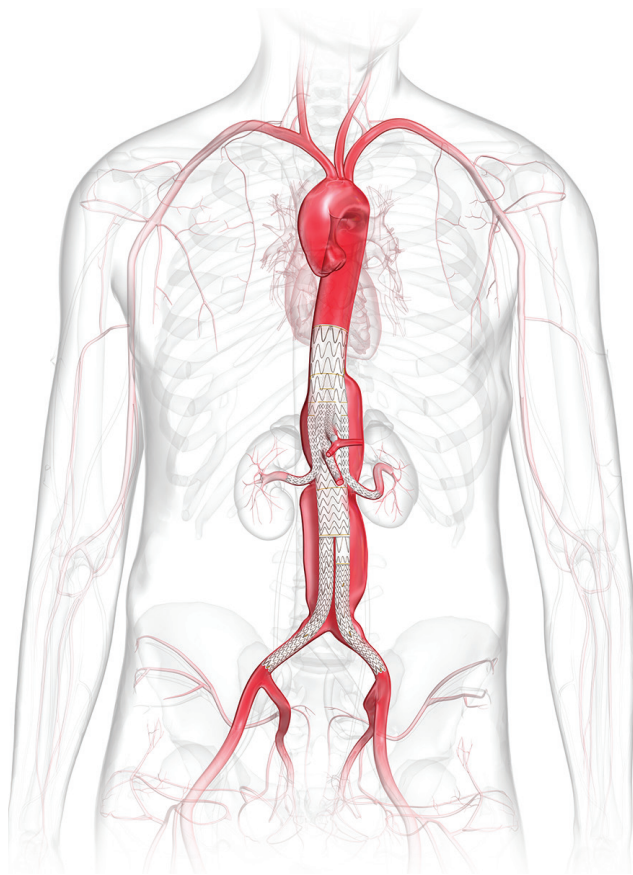


Step 1

First stage TAMBE
Aortic Component
deployment.

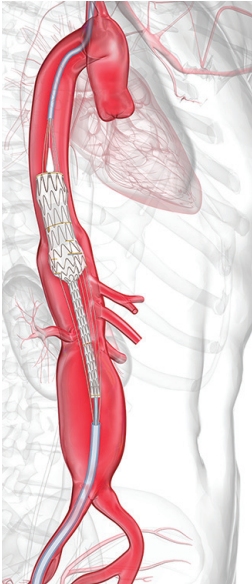


Step 2

Unconstrain
proximal anchors
and cannulate
branch vessels.

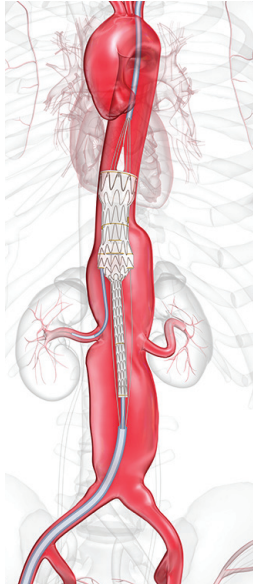


Step 3

Second stage TAMBE
Aortic Component
deployment.

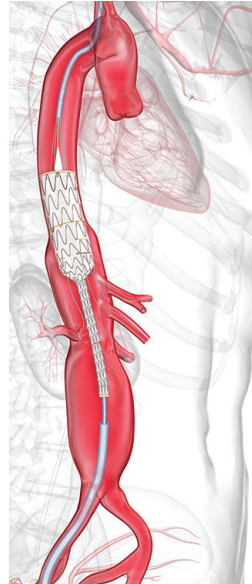


Step 4

Deploy Branch
Component in
3 of 4 branches.

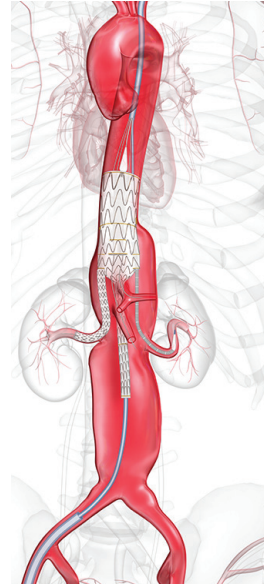


Step 5


Final stage
TAMBE Aortic
Component
deployment.

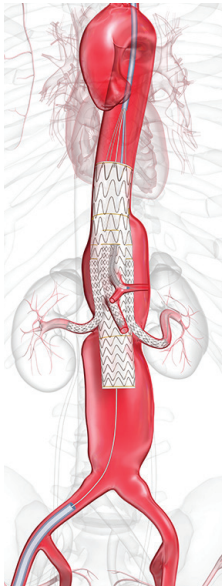**Step 6**


Deploy Branch
Component
in remaining
branch.

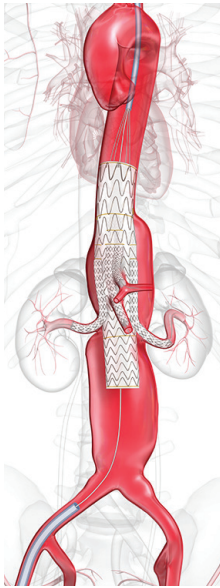**Step 7**


First stage
Distal Bifurcated
Component (DBC)
deployment.

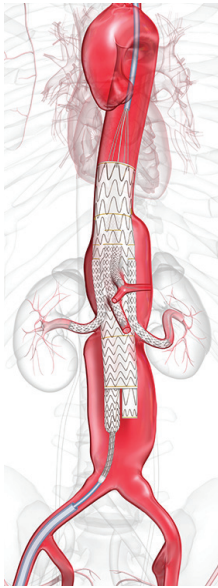**Step 8**

Contralateral
Leg Component
deployment and
extension as
necessary.

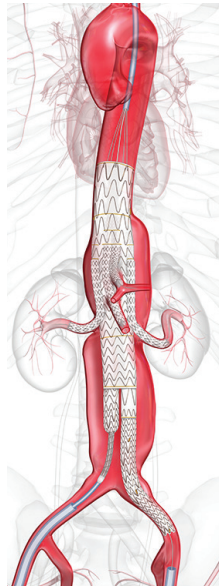**Step 9**

Final stage DBC
deployment and
extension as
necessary. Balloon
seal zones.

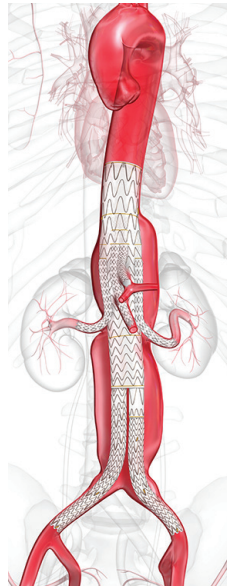

Branch Component

GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis

Catalogue number	Catheter length (cm)	Nominal stent diameter (mm)	Crimped stent length (mm)	Inflation	Inflation pressure (atm)	Final stent ID (mm)	Introducer sheath size (Fr)
BXA053902A	135	5	39	Minimum Nominal Rated Burst <i>Max post-dilated stent diameter</i>	7 11 15 8.0	4.5 5.0 5.4 8.0	7
BXA055902A	135	5	59	Minimum Nominal Rated Burst <i>Max post-dilated stent diameter</i>	7 11 15 8.0	4.5 5.0 5.3 8.0	7
BXA057902A	135	5	79	Minimum Nominal Rated Burst <i>Max post-dilated stent diameter</i>	6 12 15 8.0	4.3 5.0 5.3 8.0	7
BXA063902A	135	6	39	Minimum Nominal Rated Burst <i>Max post-dilated stent diameter</i>	7 14 15 8.0	5.0 6.0 6.1 8.0	7
BXA065902A	135	6	59	Minimum Nominal Rated Burst <i>Max post-dilated stent diameter</i>	7 14 15 8.0	5.1 6.0 6.1 8.0	7
BXA067902A	135	6	79	Minimum Nominal Rated Burst <i>Max post-dilated stent diameter</i>	7 12 15 8.0	5.3 6.0 6.3 8.0	7
BXA073902A	135	7	39	Minimum Nominal Rated Burst <i>Max post-dilated stent diameter</i>	6 12 15 11.0	6.0 7.0 7.3 11.0	7

Catalogue number	Catheter length (cm)	Nominal stent diameter (mm)	Crimped stent length (mm)	Inflation	Inflation pressure (atm)	Final stent ID (mm)	Introducer sheath size (Fr)
BXA075902A	135	7	59	Minimum	6	6.0	7
				Nominal	12	7.0	
				Rated Burst	15	7.3	
				<i>Max post-dilated stent diameter</i>			11.0
BXA077902A	135	7	79	Minimum	6	6.0	7
				Nominal	14	7.0	
				Rated Burst	15	7.1	
				<i>Max post-dilated stent diameter</i>			11.0
BXA083902A	135	8	39	Minimum	6	6.7	7
				Nominal	12	8.0	
				Rated Burst	13	8.1	
				<i>Max post-dilated stent diameter</i>			11.0
BXA085902A	135	8	59	Minimum	6	6.8	7
				Nominal	12	8.0	
				Rated Burst	13	8.1	
				<i>Max post-dilated stent diameter</i>			11.0
BXA087902A	135	8	79	Minimum	6	7.0	8
				Nominal	11	8.0	
				Rated Burst	13	8.3	
				<i>Max post-dilated stent diameter</i>			11.0
BXA093902A	135	9	39	Minimum	7	8.0	8
				Nominal	12	9.0	
				Rated Burst	13	9.1	
				<i>Max post-dilated stent diameter</i>			13.0
BXA095902A	135	9	59	Minimum	6	7.8	8
				Nominal	11	8.9	
				Rated Burst	13	9.2	
				<i>Max post-dilated stent diameter</i>			13.0
BXA097902A	135	9	79	Minimum	5	7.8	8
				Nominal	10	8.9	
				Rated Burst	13	9.4	
				<i>Max post-dilated stent diameter</i>			13.0

Distal Bifurcated Component

GORE® EXCLUDER® Iliac Branch Endoprosthesis

Catalogue number	Proximal device diameter (mm)	Distal device diameter (mm)	Intended iliac vessel diameter (mm)	Overall device length (cm)	Length to contralateral leg hole (cm)	Recommended GORE® DRYSEAL Flex Introducer Sheath (Fr)
CEB231010A	23	10	8-9	10	5.5	16
CEB231210A	23	12	10-11	10	5.5	16
CEB231410A	23	14.5	12-13.5	10	5.5	16

Contralateral Leg Endoprosthesis

GORE® EXCLUDER® AAA Endoprosthesis


Catalogue number	Intended iliac inner diameter (mm)	Iliac Endoprosthesis diameter (mm)	Contralateral leg length (cm)	Recommended GORE® DRYSEAL Flex Introducer Sheath (Fr)
PLC121000	10–11	12	10	12
PLC121200	10–11	12	12	12
PLC121400	10–11	12	14	12
PLC141000	12–13.5	14.5	10	12
PLC141200	12–13.5	14.5	12	12
PLC141400	12–13.5	14.5	14	12
PLC161000*	13.5–14.5	16	9.5	12
PLC161200*	13.5–14.5	16	11.5	12
PLC161400*	13.5–14.5	16	13.5	12
PLC181000*	14.5–16.5	18	9.5	12
PLC181200*	14.5–16.5	18	11.5	12
PLC181400*	14.5–16.5	18	13.5	12
PLC201000*	16.5–18.5	20	9.5	12
PLC201200*	16.5–18.5	20	11.5	12
PLC201400*	16.5–18.5	20	13.5	12
PLC231000*	18.5–21.5	23	10	14
PLC231200*	18.5–21.5	23	12	14
PLC231400*	18.5–21.5	23	14	14
PLC271000*	21.5–25.0	27	10	15
PLC271200*	21.5–25.0	27	12	15
PLC271400*	21.5–25.0	27	14	15

* PLEASE NOTE: All large diameter Contralateral Leg Endoprostheses (16/18/20/23/27 mm) can be used as Iliac Extenders.

Iliac Extender Endoprosthesis

GORE® EXCLUDER® AAA Endoprosthesis

Catalogue number	Intended iliac inner diameter (mm)	Endoprosthesis diameter (mm)	Endoprosthesis length (cm)	Recommended GORE® DRYSEAL Flex Introducer Sheath (Fr)
PLL161007	8–9	10	7	12
PLL161207	10–11	12	7	12
PLL161407	12–13.5	14.5	7	12

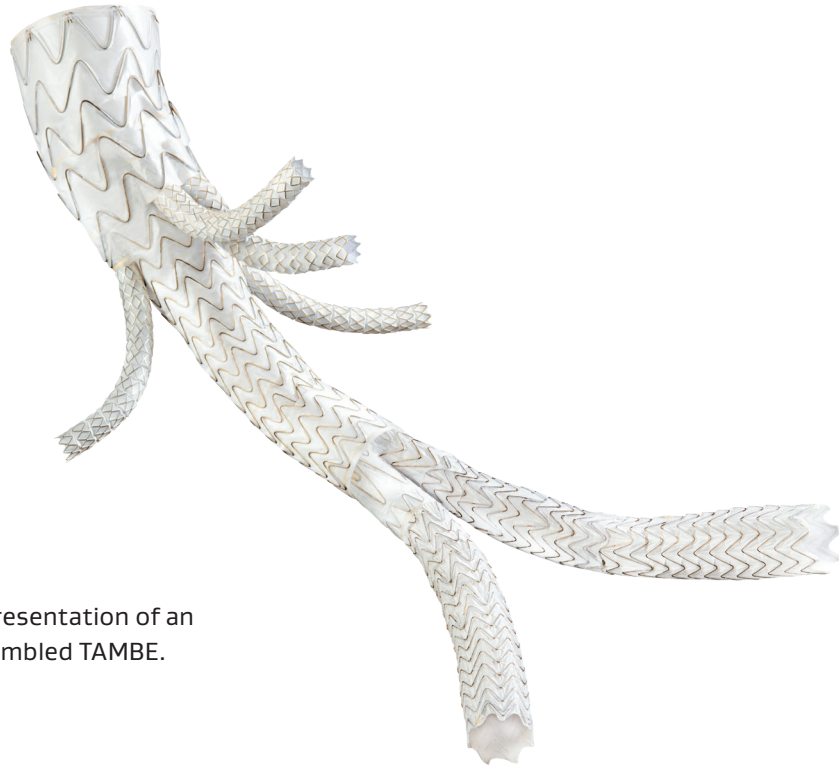

Representation of an
assembled TAMBE.

GORE® DRYSEAL Flex Introducer Sheath

Catalogue number	Sheath size (Fr)	Minimum sheath ID (mm)	Nominal sheath OD (mm)	Working length (cm)
DSF1233	12	4.0	4.7	33
DSF1245	12	4.0	4.7	45
DSF1433	14	4.7	5.3	33
DSF1533	15	5.0	5.6	33
DSF1633	16	5.3	6.1	33
DSF1833	18	6.0	6.7	33
DSF2033	20	6.7	7.5	33
DSF2233	22	7.3	8.2	33

Aortic accessories

Gore® Tri-Lumen Catheter

Catalogue number	Size (Fr)	Length (cm)	Compatibility of central lumen (inches)	Compatibility of auxiliary lumens (inches)
TLC140	7.5	140	0.035"	0.018"

Gore® Molding & Occlusion Balloon

Catalogue number	Size (Fr)	Balloon diameter (mm)	Catheter length (cm)	Balloon length (cm)
MOB37	10	10–37	90	4

Gore® Tri-Lobe Balloon Catheter

Catalogue number	Size (Fr)	Vessel diameter (mm)	Catheter length (cm)
BCM1634	18	16–34	104
BCL2645	18	26–42	104



Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where these devices, components and accessories are available. ^{Rx} Only

Products listed may not be available in all markets.

GORE, *Together, improving life*, EXCLUDER, VIABAHN and designs are trademarks of W. L. Gore & Associates.

© 2019, 2024 W. L. Gore & Associates, Inc. 241344210-EN FEBRUARY 2024

W. L. Gore & Associates, Inc.
goremedical.com

Asia Pacific +65 67332882 **Australia/New Zealand** 1800 680 424 **Europe** 00800 6334 4673
United States Flagstaff, AZ 86004 800 437 8181 928 779 2771

