



# Measurement / Device selection form

Patient ID: \_\_\_\_\_

Institution: \_\_\_\_\_

Physician: \_\_\_\_\_

Imaging date: \_\_\_\_\_


Lowest renal artery:  Right  Left  Same

Lowest renal c-arm angle: \_\_\_\_\_°

Access side:  Right  Left

Neck angle(s): \_\_\_\_\_° Neck length: \_\_\_\_\_ Neck diameter: at 0 mm \_\_\_\_\_ at 5 mm \_\_\_\_\_ at 10 mm \_\_\_\_\_ at 15 mm \_\_\_\_\_

Length of lowest renal to aortic bifurcation: \_\_\_\_\_

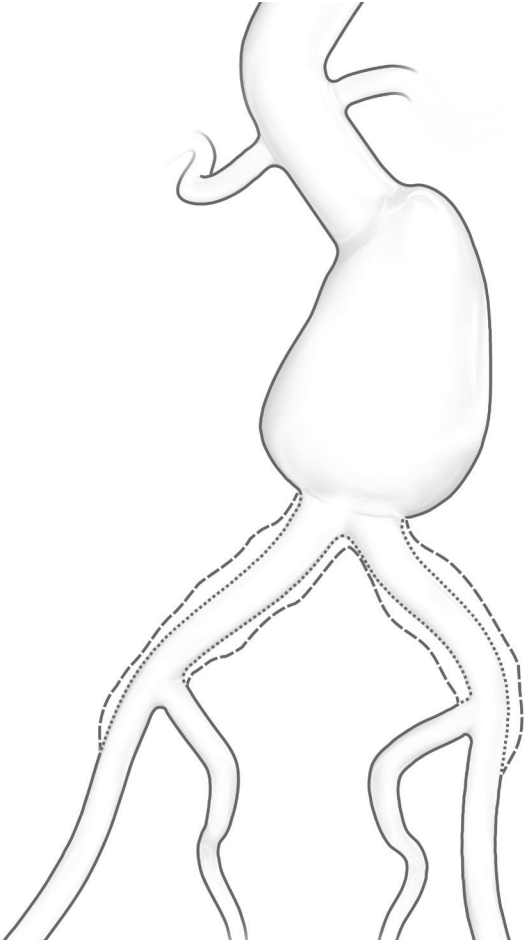

Lowest renal to right IIA length: \_\_\_\_\_

Lowest renal to left IIA length: \_\_\_\_\_

Right IIA c-arm angles: \_\_\_\_\_

Left IIA c-arm angles: \_\_\_\_\_

Right common iliac length: \_\_\_\_\_

Left common iliac length: \_\_\_\_\_

Right common iliac diameter: \_\_\_\_\_

Left common iliac diameter: \_\_\_\_\_

Access diameter: \_\_\_\_\_

Access diameter: \_\_\_\_\_

### GORE® EXCLUDER® Iliac Branch Endoprosthesis

Minimum right common iliac diameter: \_\_\_\_\_

Minimum left common iliac diameter: \_\_\_\_\_

Right internal iliac diameter: \_\_\_\_\_

Left internal iliac diameter: \_\_\_\_\_

Right external iliac diameter: \_\_\_\_\_

Left external iliac diameter: \_\_\_\_\_

Refer to the GORE® EXCLUDER® AAA Endoprosthesis, GORE® EXCLUDER® Conformable AAA Endoprosthesis and GORE® EXCLUDER® Iliac Branch Endoprosthesis *Instructions for Use* for approved indications, directions and warnings. If the physician's proposed use of the device is outside the scope of the *Instructions for Use*, Gore is providing this information in response to the physician inquiry. Physician, as a medical professional, is responsible for evaluating the information provided and deciding on the use of the device.

# Endovascular aortic configurations

## Gore® EXCLUDER® AAA Endoprosthesis featuring C3® Delivery System

### Trunk-Ipsilateral Leg Endoprosthesis

Catalogue number	Intended aortic inner diameter (mm)	Aortic endoprosthesis diameter (mm)	Intended iliac inner diameter (mm)	Iliac endoprosthesis diameter (mm)	Overall device length (cm)	Recommended sheath size (Fr)
RLT231212	19-21	23	10-11	12	12	16
RLT231214	19-21	23	10-11	12	14	16
RLT231216	19-21	23	10-11	12	16	16
RLT231218	19-21	23	10-11	12	18	16
RLT231412	19-21	23	12-13.5	14.5	12	16
RLT231414	19-21	23	12-13.5	14.5	14	16
RLT231416	19-21	23	12-13.5	14.5	16	16
RLT231418	19-21	23	12-13.5	14.5	18	16
RLT261212	22-23	26	10-11	12	12	16
RLT261214	22-23	26	10-11	12	14	16
RLT261216	22-23	26	10-11	12	16	16
RLT261218	22-23	26	10-11	12	18	16
RLT261412	22-23	26	12-13.5	14.5	12	16
RLT261414	22-23	26	12-13.5	14.5	14	16
RLT261416	22-23	26	12-13.5	14.5	16	16
RLT261418	22-23	26	12-13.5	14.5	18	16
RLT281212	24-26	28.5	10-11	12	12	18
RLT281214	24-26	28.5	10-11	12	14	18
RLT281216	24-26	28.5	10-11	12	16	18
RLT281218	24-26	28.5	10-11	12	18	18
RLT281412	24-26	28.5	12-13.5	14.5	12	18
RLT281414	24-26	28.5	12-13.5	14.5	14	18
RLT281416	24-26	28.5	12-13.5	14.5	16	18
RLT281418	24-26	28.5	12-13.5	14.5	18	18
RLT311413	27-29	31	12-13.5	14.5	13	18
RLT311415	27-29	31	12-13.5	14.5	15	18
RLT311417	27-29	31	12-13.5	14.5	17	18
RLT351414	30-32	35	12-13.5	14.5	14	18
RLT351416	30-32	35	12-13.5	14.5	16	18
RLT351418	30-32	35	12-13.5	14.5	18	18

### Contralateral Leg Endoprosthesis

Catalogue number	Intended iliac inner diameter (mm)	Iliac endoprosthesis diameter (mm)	Contralateral Leg length (cm)	Recommended sheath size (Fr)
PLC121000	10-11	12	10	12
PLC121200	10-11	12	12	12
PLC121400	10-11	12	14	12
PLC141000	12-13.5	14.5	10	12
PLC141200	12-13.5	14.5	12	12
PLC141400	12-13.5	14.5	14	12
PLC161000*	13.5-14.5	16	9.5	12
PLC161200*	13.5-14.5	16	11.5	12
PLC161400*	13.5-14.5	16	13.5	12
PLC181000*	14.5-16.5	18	9.5	12
PLC181200*	14.5-16.5	18	11.5	12
PLC181400*	14.5-16.5	18	13.5	12
PLC201000*	16.5-18.5	20	9.5	12
PLC201200*	16.5-18.5	20	11.5	12
PLC201400*	16.5-18.5	20	13.5	12
PLC231000*	18.5-21.5	23	10	14
PLC231200*	18.5-21.5	23	12	14
PLC231400*	18.5-21.5	23	14	14
PLC271000*	21.5-25.0	27	10	15
PLC271200*	21.5-25.0	27	12	15
PLC271400*	21.5-25.0	27	14	15

\* PLEASE NOTE: All large diameter Contralateral Leg Endoprostheses (16, 18, 20, 23, 27 mm) can be used as Iliac Extenders.

### Iliac Extender Endoprosthesis

Catalogue number	Intended iliac inner diameter (mm)	Endoprosthesis diameter (mm)	Endoprosthesis length (cm)	Recommended sheath size (Fr)
PLL161007	8-9	10	7	12
PLL161207	10-11	12	7	12
PLL161407	12-13.5	14.5	7	12

### Aortic Extender Endoprosthesis

Catalogue number	Intended aortic inner diameter (mm)	Endoprosthesis diameter (mm)	Endoprosthesis length (cm)	Recommended sheath size (Fr)
PLA230300	19-21	23	3.3	16
PLA260300	22-23	26	3.3	16
PLA280300	24-26	28.5	3.3	16
PLA320400	27-29	32	4.5	17
PLA360400	30-32	36	4.5	18

## Gore® EXCLUDER® Conformable AAA Endoprosthesis with ACTIVE CONTROL System

### Trunk-Ipsilateral Leg Endoprosthesis

Catalogue number	Intended aortic inner diameter (mm)	Aortic endoprosthesis diameter (mm)	Intended iliac inner diameter (mm)	Iliac endoprosthesis diameter (mm)	Overall device length (cm)	Recommended sheath size (Fr)
CXT201212	16-18	20	10-11	12	12	15
CXT201412	16-18	20	12-13.5	14.5	12	15
CXT231212	19-21	23	10-11	12	12	15
CXT231412	19-21	23	12-13.5	14.5	12	15
CXT261212	22-23	26	10-11	12	12	16
CXT261412	22-23	26	12-13.5	14.5	12	16
CXT281212	24-26	28.5	10-11	12	12	16
CXT281412	24-26	28.5	12-13.5	14.5	12	16
CXT321414	27-29	32	12-13.5	14.5	14	18
CXT361414	30-32	36	12-13.5	14.5	14	18

### Aortic Extender Endoprosthesis

Catalogue number	Intended aortic inner diameter (mm)	Endoprosthesis diameter (mm)	Endoprosthesis length (cm)	Recommended sheath size (Fr)
CXA200005	16-18	20	4.5	15
CXA230005	19-21	23	4.5	15
CXA260005	22-23	26	4.5	15
CXA280005	24-26	28.5	4.5	16
CXA320005	27-29	32	4.5	18
CXA360005	30-32	36	4.5	18

## Gore® EXCLUDER® Iliac Branch Endoprosthesis

### Iliac Branch Component

Catalogue number	Intended external iliac vessel diameter (mm)	Diameter iliac end (mm)	Iliac branch component length (cm)	Recommended sheath size (Fr)
CEB231010A	6.5-9	10	10	16
CEB231210A	10-11	12	10	16
CEB231410A	12-13.5	14.5	10	16

### Internal Iliac Component

Catalogue number	Intended internal iliac vessel diameter (mm)	Diameter iliac end (mm)	Internal iliac component length (cm)	Recommended sheath size (Fr x cm)
HGB161007A	6.5-9	10	7	12 x 45
HGB161207A	10-11	12	7	12 x 45
HGB161407A	12-13.5	14.5	7	12 x 45

## Aortic accessories

### Gore® DrySeal Flex Introducer Sheath

Catalogue number	Sheath size (Fr)	Minimum sheath ID (mm)	Nominal sheath OD (mm)	Working length (cm)
DSF1033	10	3.3	4.0	33
DSF1045	10	3.3	4.0	45
DSF1233	12	4.0	4.7	33
DSF1245	12	4.0	4.7	45
DSF1433	14	4.7	5.3	33
DSF1533	15	5.0	5.6	33
DSF1633	16	5.3	6.1	33
DSF1833	18	6.0	6.7	33

### Gore® Molding & Occlusion Balloon Catheter

Catalogue number	Size (Fr)	Balloon diameter (mm)	Catheter length (cm)
MOB37	10	10-37	90

### Gore® Tri-Lobe Balloon Catheter

Catalogue number	Size (Fr)	Vessel diameter (mm)	Catheter length (cm)
BCM1634	18	16-32	104
BCL2645	18	26-42	104

## Notes

Consult Instructions for Use  
[eifu.goremedical.com](http://eifu.goremedical.com)

Refer to *Instructions for Use* at [eifu.goremedical.com](http://eifu.goremedical.com) for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. ® only

Products listed may not be available in all markets.

GORE, EXCLUDER and designs are trademarks of W. L. Gore & Associates.  
 © 2019, 2020 W. L. Gore & Associates, Inc. 2026108-EN DECEMBER 2020

