


INSTRUCTIONAL LOADING CARD

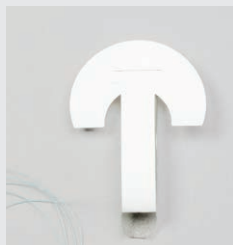

Cartridge sleeve



Anvil sleeve


This product has been designed to be used with the stapler types listed on the product labeling. If used with other staplers not identified on the product labeling, potential problems include but are not limited to:

- The device may not fit the stapler jaw, resulting in difficulty in loading, positioning, conforming to the stapler jaw, or releasing the GORE® SEAMGUARD® Bioabsorbable Staple Line Reinforcement material.
- All of the staple holes and divots within stapler's tissue resection area may not be covered.
- Proper staple formation may be affected if there is insufficient staple size.




1

Slide the GORE® SEAMGUARD® Reinforcement sleeves onto the appropriate stapler jaws. Position the bioabsorbable material on the inside of the stapler jaws, ensuring all of the staple holes and divots along both sides of the transection blade path are covered with the bioabsorbable material.

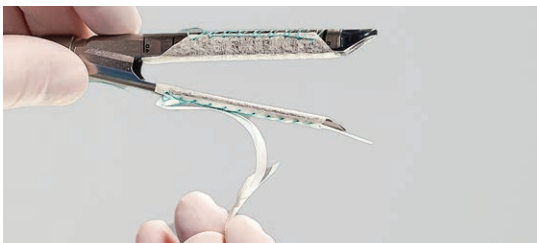

2

To remove the TYVEK® Inserts, grasp the curved or arrowed TYVEK® Insert sections and pull to separate from the underlying tabs. Orient the TYVEK® Insert parallel to the stapler shaft. Pull the insert to remove it from the device.

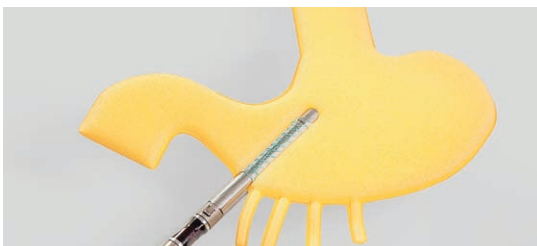

3

After stapler insertion, confirm the GORE® SEAMGUARD® Bioabsorbable Reinforcement still fully covers all the staple holes and divots. Ensure that the suture material is not inside the stapler jaw when positioning and closing the stapler. Before firing, pull the suture to release the GORE® SEAMGUARD® Reinforcement devices from the stapler jaws and discard suture. Fire the stapler.

NOTE:

- The order of cartridge and anvil loading may be altered to suit the preference of the clinician. In addition, the clinician may prefer to separate the TYVEK® Insert from the underlying tab prior to inserting onto the stapler.
- To facilitate removal of the TYVEK® Inserts, the clinician may choose to load both GORE® SEAMGUARD® Reinforcement sleeves onto the stapler jaws, close the stapler jaws, then remove the TYVEK® Inserts.

Designed to work with a variety of staplers

Configured for robotic endoscopic staplers

| Stapling device | Cartridge color | Catalogue number |
|--------------------------------|-----------------|----------------------------|
| INTUITIVE® SUREFORM 60 Stapler | Green, blue | 1BSGX160GB 12BSGX160GB* |
| INTUITIVE® SUREFORM 60 Stapler | Black | 1BSGX160BK 12BSGX160BK* |

Configured for endoscopic staplers

| Stapling device | Cartridge color | Catalogue number |
|---|---------------------------------|----------------------------|
| ETHICON ECHELON ENDOPATH® 45 Stapler | White, blue, green, gold | 1BSGEC45 12BSGEC45* |
| ETHICON ECHELON ENDOPATH® 60 Stapler | White, blue, gold, green, black | 1BSGEC60 12BSGEC60* |
| ETHICON ENDOPATH® EZ 45 Stapler | Blue, green | 1BSGEZ45 12BSGEZ45* |
| ETHICON ENDOPATH® ETS Flex 45 Stapler | White, blue, green | 1BSGETS45 12BSGETS45* |
| ETHICON ENDOPATH® ETS Flex 60 Stapler | White, blue, green | 1BSGETS60 12BSGETS60* |
| ETHICON ECHELON FLEX 45 ENDOPATH® Stapler | White, blue, gold, green | 1BSGEC45 12BSGEC45* |
| ETHICON ECHELON FLEX 60 ENDOPATH® Stapler | White, blue, gold, green, black | 1BSGEC60A 12BSGEC60A* |
| MEDTRONIC ENDO GIA Universal Roticulator 45 Stapler | White, blue | 1BSGUSS45B 12BSGUSS45B* |
| MEDTRONIC ENDO GIA Universal Roticulator 45 Stapler | Green | 1BSGUSS45G 12BSGUSS45G* |
| MEDTRONIC ENDO GIA Universal Roticulator 60 Stapler | White, blue | 1BSGUSS60B 12BSGUSS60B* |
| MEDTRONIC ENDO GIA Universal Roticulator 60 Stapler | Green | 1BSGUSS60G 12BSGUSS60G* |

Configured specifically for the MEDTRONIC ENDO GIA Reloads with TRI-STAPLE Technology

| | | |
|--|--------|----------------------------|
| MEDTRONIC ENDO GIA Ultra Universal Stapler | Purple | 1BSGTRI45P 12BSGTRI45P* |
| MEDTRONIC ENDO GIA Ultra Universal Stapler | Purple | 1BSGTRI60P 12BSGTRI60P* |
| MEDTRONIC ENDO GIA Ultra Universal Stapler | Black | 1BSGTRI45B 12BSGTRI45B* |
| MEDTRONIC ENDO GIA Ultra Universal Stapler | Black | 1BSGTRI60B 12BSGTRI60B* |

* Twelve packages per box.

 Consult Instructions
for Use
eifu.goremedical.com

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. Rx only

Products listed may not be available in all markets.

ETHICON, ECHELON ENDOPATH and ECHELON FLEX are trademarks of Ethicon, Inc. a Johnson & Johnson Company. INTUITIVE and SUREFORM are trademarks of Intuitive Surgical, Inc. MEDTRONIC, ENDO GIA and TRI-STAPLE are trademarks of Medtronic, Inc. TYVEK is a trademark of Dupont Safety and Construction, Inc.

GORE, *Together, Improving Life*, SEAMGUARD and designs are trademarks of W. L. Gore & Associates.

© 2023 W. L. Gore & Associates, Inc. 231211952-EN OCTOBER 2023

W. L. Gore & Associates, Inc.
goremedical.com

Asia Pacific +65 67332882 Europe 00800 6334 4673
United States Flagstaff, AZ 86003 800 437 8181 928 779 2771

