


GORE® Tri-Lobe Balloon Catheter Inflation Volume Guide



TRI-LOBE

BALLOON CATHETER

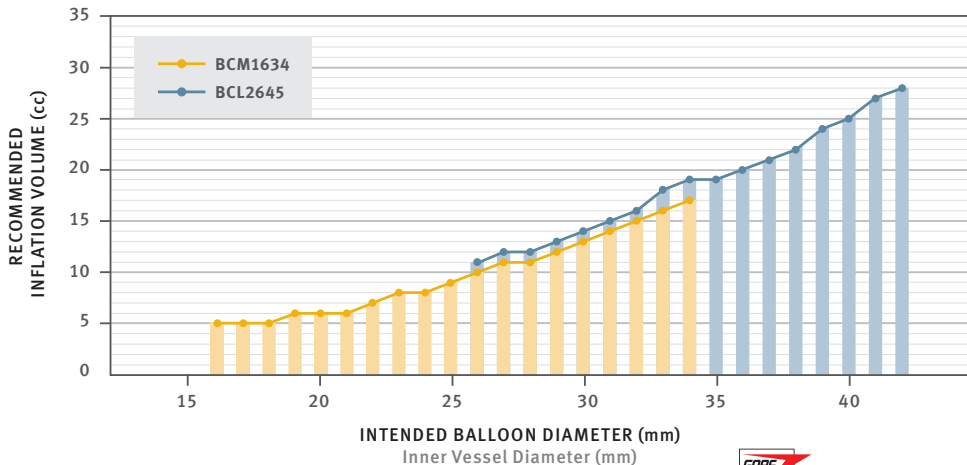

Products listed may not be available in all markets.

GORE® and designs are trademarks of W. L. Gore & Associates.
 © 2010, 2013 W. L. Gore & Associates, Inc.
 AP0128-EN3 SEPTEMBER 2013

INTENDED BALLOON DIAMETER (mm)	RECOMMENDED INFLATION VOLUME (cc)	
	BCM1634	BCL2645
Inner Vessel Diameter (mm)		
16	5	
17	5	
18	5	
19	6	
20	6	
21	6	
22	7	
23	8	
24	8	
25	9	
26	10	11
27	11	12
28	11	12
29	12	13
30	13	14
31	14	15
32	15	16
33	16	18
34	17	19
35		19
36		20
37		21
38		22
39		24
40		25
41		27
42		28