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1. Ohki T, Kichikawa K, Yokoi H, et al. Outcomes of the Japanese multicenter Viabahn trial of endovascular stent grafting for superficial femoral artery lesions. *Journal of Vascular Surgery* 2017;66(1):130-142.e1.
2. Iida O, Takahara M, Soga Y, et al; VANQUISH Investigators. One-year outcomes of heparin-bonded stent-graft therapy for real-world femoropopliteal lesions and the association of patency with the prothrombotic state based on the prospective, observational, multicenter Viabahn Stent-Graft Placement for Femoropopliteal Diseases Requiring Endovascular Therapy (VANQUISH) Study. *Journal of Endovascular Therapy* 2021;28(1):123-131.
3. Reijnen MMPJ, van Walraven LA, Fritschy WM, et al. 1-year results of a multicenter randomized controlled trial comparing heparin-bonded endoluminal to femoropopliteal bypass. *JACC: Cardiovascular Interventions* 2017;10(22):2320-2331. open access <http://www.sciencedirect.com/science/article/pii/S1936879817319775>
4. Saxon RR, Chervu A, Jones PA, et al. Heparin-bonded, expanded polytetrafluoroethylene-lined stent graft in the treatment of femoropopliteal artery disease: 1-year results of the VIPER (Viabahn Endoprosthesis with Heparin Bioactive Surface in the Treatment of Superficial Femoral Artery Obstructive Disease) trial. *Journal of Vascular & Interventional Radiology* 2013;24(2):165-173.
5. Lammer J, Zeller T, Haussegger KA, et al. Heparin-bonded covered stents versus bare-metal stents for complex femoropopliteal artery lesions: the randomized VIASTAR trial (Viabahn endoprosthesis with PROPATEN bioactive surface [VIA] versus bare nitinol stent in the treatment of long lesions in superficial femoral artery occlusive disease). *Journal of the American College of Cardiology* 2013;62(15):1320-1327. <https://www.sciencedirect.com/science/article/pii/S0735109713025941>
6. Zeller T, Peeters P, Bosiers M, et al. Heparin-bonded stent-graft for the treatment of TASC II C and D femoropopliteal lesions: the Viabahn-25 cm Trial. *Journal of Endovascular Therapy* 2014;21(6):765-774.
7. Golchehr B, Lensvelt MMA, Fritschy WM, et al. Outcome of thrombolysis and thrombectomy for thrombosed endografts inserted in the superficial femoral artery for occlusive disease. *Journal of Endovascular Therapy* 2013;20(6):836-843.

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© 2022 W. L. Gore & Associates, Inc. / 日本ゴア合同会社 21168750-JA APRIL 2022