

GORE® EXCLUDER®

Thoracoabdominal Branch
Endoprosthesis

BRIDGING THE GAP

between thoracic
and abdominal care

GET STARTED 



Together, improving life




MDR CE MARK-APPROVED,
off-the-shelf endovascular
solution for a range of
complex visceral
aortic cases.


Meet the component you were looking for

Modular design and studied compatibility
with other Gore commercial devices.


[WATCH DEPLOYMENT](#) 

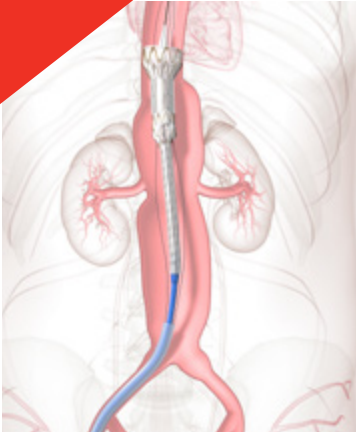


Aortic Component positioned and partially deployed.

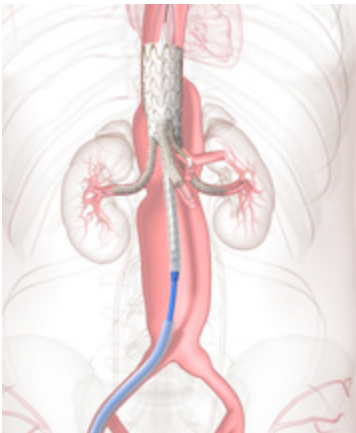

Branch Components delivered and deployed through pre-cannulated portals of Aortic Component.

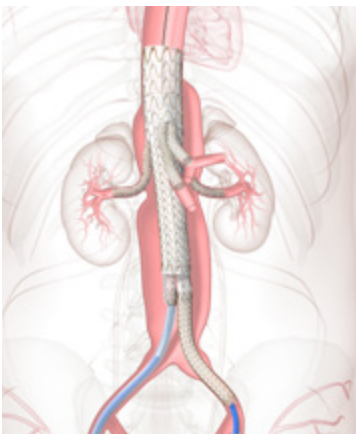

Aortic Component deployment completed, followed by **Distal Bifurcated Component** and **Contralateral Leg Component**.

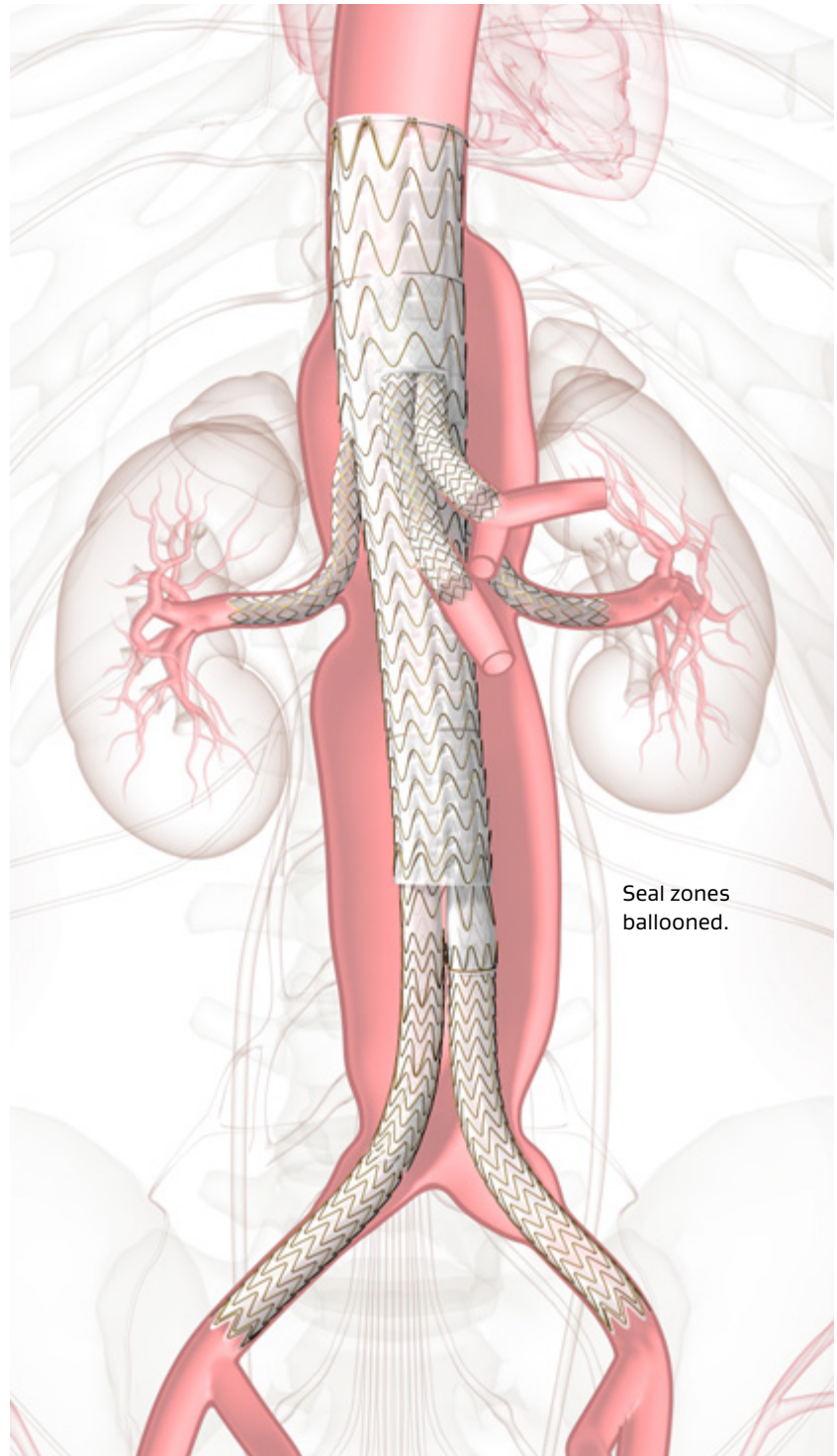

Seal zones
ballooned.

On-label clinically proven endovascular repair

Approved for use in repair of thoracoabdominal aortic aneurysms and pararenal aortic aneurysms in patients with appropriate anatomy



Pivotal Study outcomes through 12 months^{1,a}

0% Lesion-related mortality

94% Freedom from aneurysm growth

92.6% Freedom from failure of device effectiveness^b

85.3% Freedom from any device component occlusions

100% Freedom from occlusion of aortic and iliac device components

95.8% Freedom from target visceral artery occlusion

Off-the-shelf availability for timely patient care

The benefits of minimally invasive care^{3,4} without the time and effort to create custom grafts.

4.0 days

Median length of stay for TAMBE repair^{2,c}

^a All results are on a subject level except for freedom from target visceral artery occlusion.

^b A compromise of a device seal zone or loss of device integrity (e.g., device kink or collapse) requiring placement of an additional stent or stent-graft

^c GORE® EXCLUDER® Thoracoabdominal Branch Endoprosthesis

Another innovation in branched technology

With dedicated support you can depend on in complex procedures.

- A deep reservoir of product knowledge.
- Our representatives support hundreds of aortic cases per year.
- Dedicated clinical specialist for in-person or remote clinical support.

Contact your Gore representative for more information.

Aortic Component GORE® EXCLUDER® Thoracoabdominal Branch Endoprosthesis

Proximal Diameter (mm) x Distal Diameter (mm) x Length (mm)

Catalogue Number	Device Size	Intended Proximal Aortic Diameter (mm)	Recommended Sheath Size (Fr)
ATAA43120160E	31 x 20 x 160	22-29	22
ATAA43720160E	37 x 20 x 160	27-34	22

GORE® Tri-Lumen Catheter

Catalogue Number	Catheter Working Length (cm)	Outer Diameter (Fr)	Introducer Sheath Compatibility (Fr)
TLC140E	140	7.5	12

References

1. Farber MA, Han S, Makaroun MS. One-year outcomes from the pivotal trial of a four-branch off-the shelf solution to treat pararenal and extent IV thoracoabdominal aortic aneurysms. *Journal of Vascular Surgery* 2025;82(3):740-749.e2.
2. Mark A Farber, Jon S Matsumura, Sukgu Han, Michel S Makaroun, Bjoern D Suckow, Carlos H Timaran, Bernardo C Mendes, Gustavo S Oderich Early outcomes from the pivotal trial of a four-branch off-the shelf solution to treat complex abdominal and type IV thoracoabdominal aortic aneurysms. *J Vasc Surg* 2024 Nov;80(5):1326-1335.e4. doi: 10.1016/j.jvs.2024.05.020. Epub 2024 Jun 20.
3. Bavaria JE, Appoo JJ, Makaroun MS, et al.; Gore TAG Investigators. Endovascular stent grafting versus open surgical repair of descending thoracic aortic aneurysms in low-risk patients: a multicenter comparative trial. *Journal of Thoracic & Cardiovascular Surgery* 2007;133(2):369-377.
4. Arko FR 3rd, Murphy EH, Boyes C, et al. Current status of endovascular aneurysm repair: 20 years of learning. *Seminars in Vascular Surgery* 2012;25(3):131-135.

 Consult Instructions for Use at eifu.goremedical.com

Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. Rx Only

Products listed may not be available in all markets.

© 2026 Gore Medical GmbH. All rights reserved. All trademarks referenced are trademarks of either a member of the Gore group of affiliated companies or their respective owners. "Together, improving life" mark and design are trademarks of a Gore company. 25AR2232-EN02 FEBRUARY 2026 MAT-0049-1

W. L. Gore & Associates, Inc.

goremedical.com

Asia Pacific +65 6733 2882 **Australia/New Zealand** 1800 680 424 **Europe** 00800 6334 4673

United States Flagstaff, AZ 86004 800 437 8181 928 779 2771 MAT-0049-1

Together, improving life

