


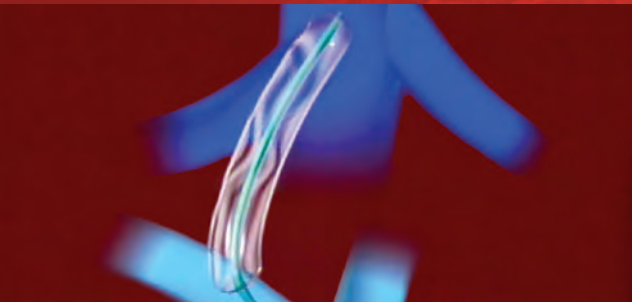


GORE® VIATORR®

TIPS Endoprosthesis


主要植入步骤



预扩张通路。可以使用较正常尺寸为小的球囊以提高触感。

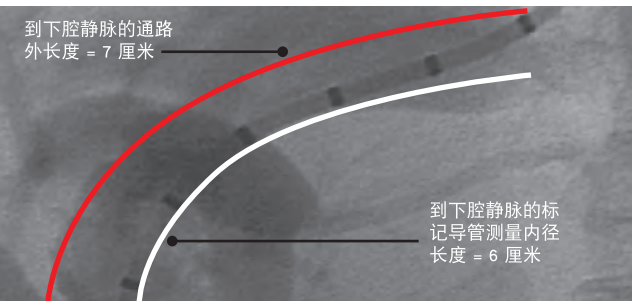

使用标记导管测量通路长度。在测量值上增加 1 厘米以选择正确的覆膜段长度。

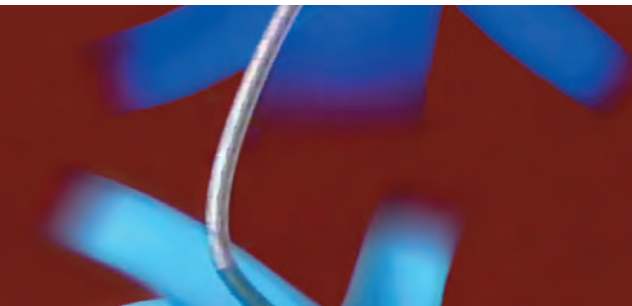

De Novo 手术
推送导引鞘进入门静脉 ≥ 3 厘米。


修复手术
将导引鞘推送至正在接受修复的支架的远端。



用手指盖住透明的入路套管头端，器械与之齐平。



将透明的入路套管插入止血阀直到感觉到停止。导管推送期间，对入路套管维持向前的压力。

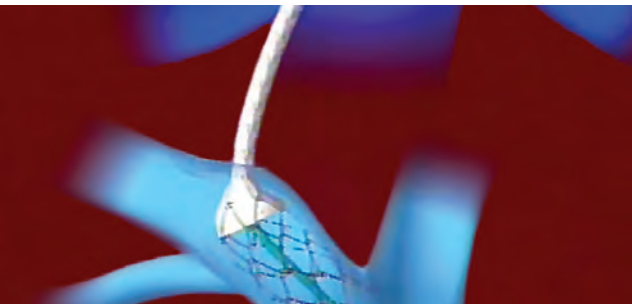

De Novo 手术
将导引鞘拉回至下腔静脉以展开裸支架部分。将植入支架轻轻往回拉直到在门静脉/肝实质交界处感觉到阻力。

修复手术
在展开裸支架部分前，将金标记环与门静脉/肝实质交界对齐。



使用 GORE SIM-PULL 输送系统展开覆膜支架部分。

注：应在术后 72 小时施行跟进超声，以获得最佳效果。

尺寸选择表

植入支架尺寸						推荐的辅助器械		
内径 (毫米)	覆膜段长度/裸段长度 (厘米 / 厘米)					最大导丝直径 (英寸)	止血导引鞘 (Fr)	最大扩张球囊 直径 (毫米)
标号	4 / 2	5 / 2	6 / 2	7 / 2	8 / 2			
8	x	x	x	x	x	≤ 0.038	10	8
10	x	x	x	x	x	≤ 0.038	10	10

 Consult Instructions for Use

INDICATIONS FOR USE IN THE US: The GORE® VIATORR® TIPS Endoprosthesis is indicated for use in the *de novo* and revision treatment of portal hypertension and its complications such as variceal bleeding, gastropathy, refractory ascites, and / or hepatic hydrothorax. **INDICATIONS FOR USE UNDER CE MARK:** The GORE® VIATORR® TIPS Endoprosthesis is indicated for use in the treatment of portal hypertension and its complications such as: variceal bleeding refractory to, or intolerant of, conventional therapies, inaccessible varices, gastropathy, refractory ascites, and / or hepatic hydrothorax. Refer to *Instructions for Use* at goremedical.com for a complete description of all contraindications, warnings, precautions and adverse events. Rx Only

适用范围：TIPS覆膜支架系统适用于初治和修复治疗门静脉高血压及其并发症，如静脉曲张出血、胃病、顽固性腹水和/或肝性胸水。

W. L. GORE & ASSOCIATES, INC.
Flagstaff, AZ 86004

+65.67332882 (Asia Pacific)
00800.6334.4673 (Europe)
800.437.8181 (United States)
928.779.2771 (United States)

goremedical.com



所列产品不一定在所有市场中均有售。
GORE®, PERFORMANCE BY DESIGN, SIM-PULL, VIATORR®, and designs are trademarks of W. L. Gore & Associates.
© 2006, 2007, 2009, 2010 W. L. Gore & Associates, Inc. AN0669-SZH2 NOVEMBER 2010

PERFORMANCE
by design