



Together, improving life

GORE® VIABAHN® VBX BALLOON EXPANDABLE ENDOPROSTHESIS SIZING TABLE

6Fr
compatible
GREATER
VERSATILITY

Stent labeled/ nominal diameter (mm)	Stent length (mm)	Introducer sheath size (Fr)	At minimal pressure			At nominal pressure			At rated burst pressure			At maximum post-dilated stent diameter ^b	
			Minimal inflation pressure (atm/kPa)	Final stent ID (mm)	Stent length ^a (mm)	Nominal inflation pressure (atm/kPa)	Final stent ID (mm)	Stent length ^a (mm)	Rated burst pressure (atm/kPa)	Final stent ID (mm)	Stent length ^a (mm)	Maximum post-dilated stent diameter (mm)	Stent length at maximum post-dilated stent diameter ^a (mm)
5	15	6	7/709	4.3	14.5	13/1,317	5.0	14.1	16/1,621	5.3	14.0	8	10.5
	19	6	6/608	4.4	18.9	11/1,115	5.0	18.6	16/1,621	5.4	18.2	8	13.8
	29	6	7/709	4.5	28.7	12/1,216	5.0	28.3	16/1,621	5.4	28.0	8	22.3
	39	6	7/709	4.6	38.9	11/1,115	5.0	38.5	16/1,621	5.5	38.1	8	32.0
	59	6	7/709	4.4	58.9	12/1,216	5.0	58.6	16/1,621	5.4	58.4	8	51.2
	79	6	7/709	4.3	79.0	14/1,419	5.0	78.9	17/1,723	5.3	78.9	8	72.1
6	15	6	6/608	5.0	14.1	18/1,824	6.0	13.5	18/1,824	6.0	13.5	8	10.5
	19	6	6/608	5.1	18.9	16/1,621	6.0	18.4	17/1,723	6.1	18.3	8	14.6
	29	6	6/608	5.1	28.4	15/1,520	6.0	28.3	17/1,723	6.1	28.2	8	24.3
	39	6	6/608	5.0	38.5	14/1,419	6.0	38.1	17/1,723	6.3	38.0	8	33.3
	59	6	6/608	5.1	58.4	15/1,520	6.0	58.6	17/1,723	6.1	58.6	8	52.3
	79	6	6/608	5.1	78.7	14/1,419	6.0	79.4	17/1,723	6.2	79.6	8	73.6
7	15	6	5/507	6.2	13.9	9/912	7.0	13.8	11/1,115	7.2	13.7	11	9.5
	19	6	5/507	6.4	18.5	8/811	7.0	18.1	11/1,115	7.4	17.7	11	12.6
	29	6	5/507	6.3	28.1	8/811	7.0	27.9	11/1,115	7.4	27.7	11	19.4
	39	6	6/608	6.5	37.8	8/811	6.9	37.7	11/1,115	7.4	37.4	11	26.6
	59	6	6/608	6.7	58.7	7/709	6.9	58.7	11/1,115	7.4	58.5	11	46.9
	79	6	6/608	6.5	78.7	8/811	6.9	78.6	11/1,115	7.4	78.5	11	68.1
8	29	7	6/608	7.2	27.2	11/1,115	7.9	27.1	15/1,520	8.4	27.0	11	23.7
	39	7	6/608	7.2	38.0	12/1,216	8.0	37.7	15/1,520	8.4	37.5	11	30.0
	59	7	6/608	7.1	58.1	12/1,216	8.0	57.8	15/1,520	8.4	57.7	11	47.3
	79	7	6/608	6.9	77.8	12/1,216	7.9	78.3	15/1520	8.4	78.5	11	69.6
8L	29	7	6/608	6.9	29.1	12/1,216	8.0	28.8	16/1,621	8.6	28.7	16 ^c	21.8
	39	7	6/608	6.9	39.3	12/1,216	8.0	39.4	16/1,621	8.5	39.5	16 ^c	26.8
	59	8	6/608	7.0	59.1	11/1,115	7.9	58.7	16/1,621	8.7	58.4	16 ^c	44.1
	79	8	6/608	7.0	79.0	12/1,216	8.0	79.4	16/1,621	8.6	79.7	16 ^c	58.6
9	29	7	5/507	7.6	27.7	13/1,317	9.0	26.6	13/1,317	9.3	26.4	13	20.4
	39	7	5/507	7.6	37.8	12/1,216	9.0	36.5	15/1,520	9.5	36.0	13	23.2
	59	7	6/608	7.9	58.2	11/1,115	9.0	57.9	15/1,520	9.6	57.7	13	38.1
	79	7	6/608	8.0	77.2	11/1,115	9.0	76.9	15/1,520	9.6	76.6	13	62.7
10	29	7	6/608	8.4	27.6	14/1,419	10.0	26.3	15/1,520	10.1	26.2	13	19.1
	39	7	6/608	8.7	37.2	12/1,216	10.0	36.6	15/1,520	10.4	36.3	13	31.1
	59	7	6/608	8.7	57.3	12/1,216	10.0	57.2	12/1,216	10.0	57.2	13	44.6
	79	7	6/608	8.8	76.7	12/1,216	10.0	77.1	12/1,216	10.0	77.1	13	68.4
11	29	8	6/608	9.5	26.9	10/1,013	11.0	26.0	11/1,115	11.2	25.8	16 ^c	17.8
	39	8	6/608	9.7	36.6	10/1,013	11.1	35.4	11/1,115	11.4	35.1	16 ^c	25.8
	59	8	6/608	9.8	56.6	9/912	11.0	55.9	11/1,115	11.5	55.4	16 ^c	42.8
	79	8	6/608	9.7	77.6	9/912	10.9	77.2	11/1,115	11.4	76.9	16 ^c	60.6

a. Based on idealized in vitro conditions and provided to be used as a general guide.
a. Secondary balloon required to post-dilate the stent beyond its nominal deployed diameter (secondary balloon not included).
c. Technical limit of the device as determined by in vitro testing for the indicated use; device expansion beyond 13 mm was not studied as part of the Gore VBX FLEX Clinical Study. See *Instructions for Use*.
1 Consult Instructions for Use at eifu.goremedical.com Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. ^{Rx} Only

INDICATIONS FOR USE IN THE US: The GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis is indicated for the treatment of de novo or restenotic lesions found in iliac arteries with reference vessel diameters ranging from 5 mm–13 mm and lesion lengths up to 110 mm, including lesions at the aortic bifurcation. The GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis is also indicated for use with thoracoabdominal and pararenal branched devices indicated with the GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis as a branch component. **CONTRAINDICATIONS:** Do not use the GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis in patients with known hypersensitivity to heparin, including those patients who have had a previous incident of Heparin-Induced Thrombocytopenia (HIT) type II.

Products listed may not be available in all markets.
© 2026 W. L. Gore & Associates, Inc. All rights reserved. All trademarks referenced are trademarks of either a member of the Gore group of affiliated companies or their respective owners.
“Together, improving life” mark and designs are trademarks of a Gore company. 26PL6011-EN01 JULY 2026 MAT-0594-2

Learn more

