

SIZING CARD

GORE® VIABAHN® VBX

Balloon Expandable
Endoprosthesis



6Fr
compatible

**GREATER
VERSATILITY**

Together, improving life



Device sizing	Introducer sheath (Fr)						Maximum post-dilated stent diameter (mm) ^a
	Stent length						
Stent labeled/ nominal diameter (mm)	15 mm	19 mm	29 mm	39 mm	59 mm	79 mm	
5	6	6	6	6	6	6	8
6	6	6	6	6	6	6	8
7	6	6	6	6	6	6	11
8	NA	NA	7	7	7	7	11
8L	NA	NA	7	7	8	8	16 ^b
9	NA	NA	7	7	7	7	13
10	NA	NA	7	7	7	7	13
11	NA	NA	8	8	8	8	16 ^b

Abbreviation: NA, not applicable. Latest VBX Stent Graft profile (Fr).

All VBX Stent Graft configurations available in 0.035" over-the-wire 80 and 135 cm catheter lengths.

a. Secondary balloon required to post-dilate the stent beyond its nominal deployed diameter (secondary balloon not included).

b. Technical limit of the device as determined by in vitro testing for the indicated use; device expansion beyond 13 mm was not studied as part of the Gore VBX FLEX Clinical Study. See *Instructions for Use*.



Refer to *Instructions for Use* at eifu.goremedical.com for a complete description of all applicable indications, warnings, precautions and contraindications for the markets where this product is available. Rx Only

INDICATIONS FOR USE IN THE US: The GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis is indicated for the treatment of de novo or restenotic lesions found in iliac arteries with reference vessel diameters ranging from 5 mm–13 mm and lesion lengths up to 110 mm, including lesions at the aortic bifurcation. The GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis is also indicated for use with thoracoabdominal and pararenal branched devices indicated with the GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis as a branch component.

CONTRAINDICATIONS: Do not use the GORE® VIABAHN® VBX Balloon Expandable Endoprosthesis in patients with known hypersensitivity to heparin, including those patients who have had a previous incident of Heparin-Induced Thrombocytopenia (HIT) type II.

Products listed may not be available in all markets.

© 2026 W. L. Gore & Associates, Inc. All rights reserved. All trademarks referenced are trademarks of either a member of the Gore group of affiliated companies or their respective owners. *"Together, improving life"* mark and designs are trademarks of a Gore company.

26PL6012-EN01 JULY 2026 MAT-0688-2

W. L. Gore & Associates, Inc.
goremedical.com

Asia Pacific +65 67332882
Australia/New Zealand 1800 680 424 **Europe** 00800 6334 4673
United States Flagstaff, AZ 86004 800 437 8181 928 779 2771

